

INGREDIENTS:

Hypochlorous Acid, Sodium Chloride, Buffer,
Purified Water

INDICATIONS:

A topical preparation for use on large and small animals to aid in cleaning and protection from environmental contaminants in minor cuts, wounds, abrasions and skin sores. Also to aid in removal of crustiness around the eyes.


STORAGE:

Store at room temperature between 59°F and 86°F (15°C – 30°C). Do not store at temperatures above 30°C. Do not store in direct sunlight.

CAUTION: Avoid storing in any uncontrolled environment such as closed vehicles in summer heat, which will cause degradation of the product.

MANUFACTURED BY:

Radix Laboratories, Inc.
1334 International Drive
Eau Claire, WI 54701
1-800-927-2349

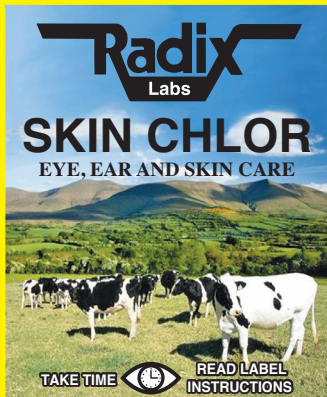

NET CONTENTS
500 ml (16 fl. oz.)

DIRECTIONS FOR USE:

Use only as directed. Formulated within an appropriate pH range to prevent discomfort when applied. Apply 3 to 4 times daily for 3 to 4 days. If redness, irritation, or swelling persists or increases, discontinue use and consult a veterinarian.

EYE CARE:

Spray onto a clean swab until saturated, then gently swab around eyes to remove crustiness and discharge.

EAR AND SKIN CARE:

Spray onto a clean swab until saturated, then gently swab affected area to remove pus and scabs. Then cover affected area(s) by direct spray.

KEEP AWAY FROM CHILDREN

FOR ANIMAL USE ONLY